



# INSTRUCTIONS OF USE

## BF001 FACEMASK

The Biomed Facemask (Bioshield-01) is entered in the ARTG, 335571.

This is a reusable antibacterial face mask that reduces the wearer's respiratory exposure to bacteria, pollen and dust. This mask was designed for personal, everyday wear and is non-sterile. It is not recommended for wear in a clinical, hospital and/or sterile environment.

### General Safety:

This mask should only be used for personal, everyday wear. It should not be used in a sterile, clinical or hospital environment.

In the event of any discomfort, pain or skin reactions while wearing the face mask, remove immediately and seek a review from your doctor.

Please ensure that you follow the washing instructions with a wash every day if you suspect you are ill or show any hay fever, cold or flu symptoms.

Else, wash every 3 days.

This mask is only suitable for use for up to 10 washes, after which it should be disposed of a day (if you are ill or show hay fever, cold or flu symptoms) or 3 days after the last wash.

Please note that this mask is intended for use as a reusable antibacterial face mask that reduces the wearer's respiratory exposure to bacteria, pollen and dust. It should not be relied on for the prevention, treatment or alleviation of any virus

Manufactured by:  
Biomed Technology Australia Pty Ltd  
438 Punchbowl Rd, Belmore NSW 2192

Top Layer: 100% cotton  
Middle layer: 65% olefin, 35% polyester  
Bottom Layer: 50% polyester, 50% copper.


## FAMILIARISE YOURSELF WITH THE MASK

Clean your hands with soap and water or hand sanitiser before touching the mask.

Remove a mask from the box and make sure there are no visible tears or holes in either side of the mask.


Determine which side of the mask is the top. The top side of the mask that has an additional horizontal seam on the outside of the mask while the bottom has a small vertical seam.

Determine which side of the mask is the front. In addition to the contours of the mask, the inside of the mask is a beige colour.

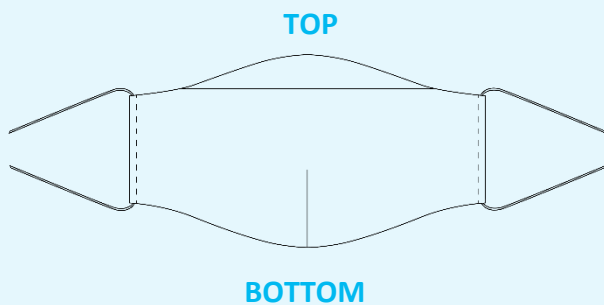


1

Hold the mask up to the face, ensuring the top section sits over the nose and the inside cream colour is facing the face.

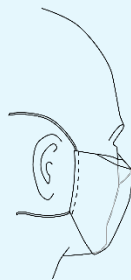


2

Tie the elastic behind the ears for a comfortable and snug fit.

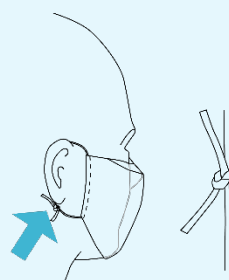

3

Ensure you cut off the excess elastic and slide the knot into the seam. Alternatively, the elastic can be tied behind the head.

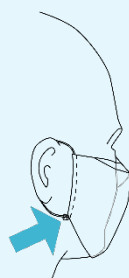

Pinch off the edge around the shape of your nose and ensure the mask covers your mouth and chin.

## HOW TO REMOVE THE MASK

Clean your hands with soap and water or hand sanitiser before touching the mask.

Avoid touching the front of the mask as it may be contaminated. Only touch the ear loops.

1. Hold both ear loops and gently lift and remove the mask.
2. Clean your hands with soap and water or hand sanitiser.
3. Please refer to the cleaning instructions below.

## HOW TO CLEAN THE MASK

Clean your hands with soap and water or hand sanitiser before and after touching the mask.

1. Submerge the mask in boiling water for 3-minutes.
2. Remove the mask and air dry.
3. Be careful not to come in contact with the boiling water to avoid burns.

### PLEASE NOTE:

You may use a neutral or mild detergent. **Do not use** a strong or abrasive cleaner or detergent, or bleach.

- **Do not** machine wash or tumble dry.
- **Do not** rub the mask during the cleaning process.
- Wash every day if you are ill or show any hay fever, cold or flu symptoms.
- Wash every 3 days if not sick.
- This mask lasts up to 10 washes.
- Dispose of 3 days (if you are not sick) or 1 day (if you are ill or show any hay fever, cold or flu symptoms) after the last wash.