

# INSTRUCTIONS OF USE

## 3 Layers Protective Mask

The 3 Layers Protective Mask is entered in the ARTG, 340188.

This is a reusable antibacterial face mask that reduces the wearer's respiratory exposure to bacteria, pollen and dust. This mask was designed for personal, everyday wear and is non-sterile. It is not recommended for wear in a clinical, hospital and/or sterile environment.

### General Safety:

This mask should only be used for personal, everyday wear. It should not be used in a sterile, clinical or hospital environment.

In the event of any discomfort, pain or skin reactions while wearing the face mask, remove immediately and seek a review from your doctor.

Please ensure that you follow the washing instructions with a wash every day if you suspect you are ill or show any hay fever, cold or flu symptoms. Else, wash every 3 days.

This mask is only suitable for use for up to 25 washes, after which it should be disposed of a day (if you are ill or show hay fever, cold or flu symptoms) or 3 days after the last wash.

Please note that this mask is intended for use as a reusable antibacterial face mask that reduces the wearer's respiratory exposure to bacteria, pollen and dust. It should not be relied on for the prevention, treatment or alleviation of any virus.

Manufactured by:  
Biomed Technology Australia Pty Ltd  
438 Punchbowl Rd, Belmore NSW 2192



### HOW TO REMOVE THE MASK

Clean your hands with soap and water or hand sanitiser before touching the mask.

Avoid touching the front of the mask as it may be contaminated. Only touch the ear loops.

1. Hold both ear loops and gently lift and remove the mask.
2. Clean your hands with soap and water or hand sanitiser.
3. Please refer to the cleaning instructions below.

### LAUNDRY GUIDE

To Correctly clean the mask:

1. Do NOT rub while washing
2. Spray 75% alcohol on both side of the masks
3. Leave it there for 15 minutes
4. Hand wash or machine wash in 3D laundry net with anti-microbial detergent
5. Ventilate it until dry; avoid sun and keep it in the shade.

### PLEASE NOTE

You may use a neutral or mild detergent. Do not use a strong or abrasive cleaner or detergent, or bleach.

- Do not machine wash or tumble dry.
- Do not rub the mask during the cleaning process.
- Wash every day if you are ill or show any hay fever, cold or flu symptoms.
- Wash every 3 days if not sick.
- **This mask lasts up to 25 washes.**
- Dispose of 3 days (if you are not sick) or 1 day (if you are ill or show any hay fever, cold or flu symptoms) after the last wash.

# FAMILIARISE YOURSELF WITH THE MASK

Clean your hands with soap and water or hand sanitiser before touching the mask.

Remove the mask from the plastic and make sure there are no visible tears or holes in either side of the mask. Determine the top by the pleats, the pleats should face down when in front of the face.

**Filtering effect (0.075µm):** 75%

**PM2.5 Protection:** Good

**Air Permeability:** Good ASTM D77-96

**Dust proof:** Yes

**Washing:** Washable 25 times

## Layers:

Outside Layer:


-100% Polyester with water repellent

Middle Layer:

-100% Polyester with water repellent

Inside Layer: (white fabric, soft to touch)

-Elastic Skin Friendly Breathable Knitted Fabric

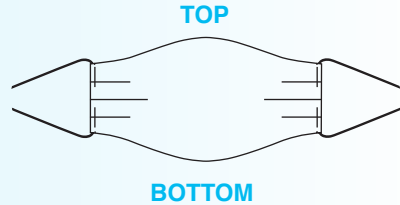


1

Hold the mask up to the face, ensuring the pleats are facing down at the front and the 'soft to touch' white layer is facing the face.



2

Tie the elastic behind the ears for a comfortable and snug fit.

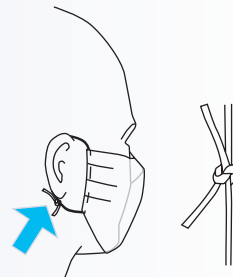

3

Ensure you cut off the excess elastic and slide the knot into the seam. Alternatively, the elastic can be tied behind the head. Pinch off the edge around the shape of your nose and ensure the mask covers your mouth and chin.

